


Export Current Protocol Test Data to Excel Spreadsheet

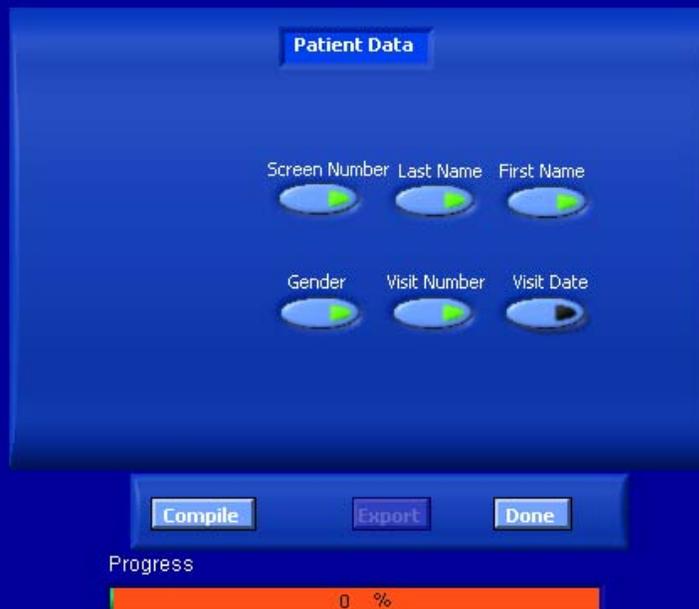

1. Select Preferences - Top Center
2. Select Patient Data - Top Center
3. Compile
4. Select Patients, Visits & Tests
5. Select Export - may have a short wait.
6. Done / or Add Additional Protocol ?
7. Add Add'l Protocol - Select protocol & repeat selection procedures.
8. Done
9. Name the Excel File
10. Take note of the location of the new file.
11. End

*** To Close without running - Select DONE
A popup will appear - Select DONE
Application will close*

**** Unabridged instructions may be found in
the User Guide - Section 3.4*